
Vocatex Standard / Vocatex Plus *Full HD*

Model: Vocatex 2014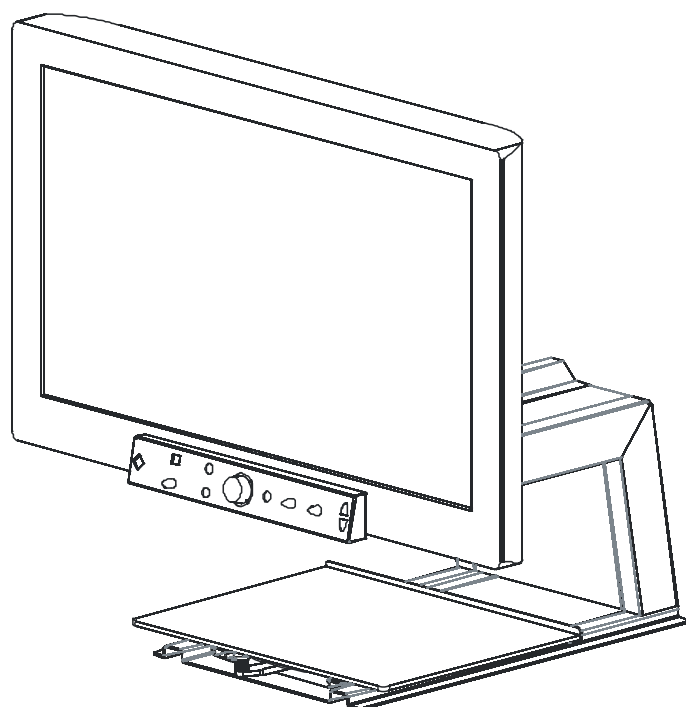

Français - Mode d'emploi - Page 1
Nederlands – Gebruikshandleiding - Pagina 11
English - User Manual - Page 22
Deutsch – Gebrauchsanweisung - Seite 32
Quick installation Guide - Page 43

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English – User Manual

This manual describes the following versions of the KOBA Vision Vocatex: Vocatex Standard, Vocatex Plus.

The latest versions of the manual can be downloaded from the KOBA Vision website: <http://www.kobavision.be>.

Release date: 11 June 2014.

Chapter 1 : Safety and maintenance instructions

Safety instructions


- Text read by the Vocatex is under no circumstance guaranteed to be a 100% correct. The text-to-speech function is solely to be used as a reading aid.
- The device is not to be used in a medical environment.
- Never sit on the sliding table.
- Never attempt to pull yourself up by holding the Vocatex.
- The power supply must only be replaced by the same type (AC Adapter Cincon TR30RAM150) after switching off the device and removing the adapter from the grid.
- The connections have been designed to only connect in 1 way. Never force a connection. The cables are only to be connected by authorised persons. Install the power supply of the device in a well ventilated area.
- Never attempt to repair the device by yourself. It is dangerous to remove protective covers which are there to prevent electric shock. You must contact KOBA Vision or your dealer in case of defects.
- To avoid electrocution, do not place the device in a spot where it can come into contact with liquids.
- When moving the device, the height adjustment must always be blocked by using the supplied orange screws.
- Consult the user manual of the screen for specific safety instructions related to the screen.
- Save this manual carefully. It contains all necessary information about your device.
- Keep the device in a dry, well-ventilated and clean place. Do not expose the device to direct sunlight or a heat source.
- Never use this device in ambient temperatures higher than 35°C.
- This device is designed as aid to magnify and to read aloud documents. Any change of purpose voids the CE-marking.
- Your product is designed and manufactured with high quality materials and components, which can be recycled and reused. Do not dispose of your old product in your general household waste bin. Dispose of the complete product (including its cables, plugs and accessories) in the designated collection facilities.

WARNING


The screen is to be assembled and disassembled by authorized persons only. Incorrect installation or assembly can cause severe damage and/or injuries.

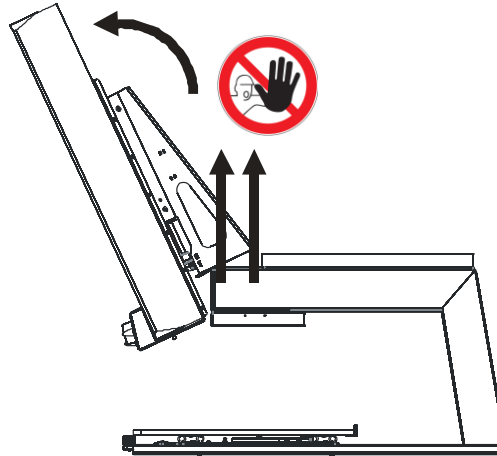WARNING

You must be extremely cautious when moving the Vocatex. The Vocatex must not be picked up by the screen, the front panel, the light casings or the slide table. Always block the height adjustment with the supplied orange screws.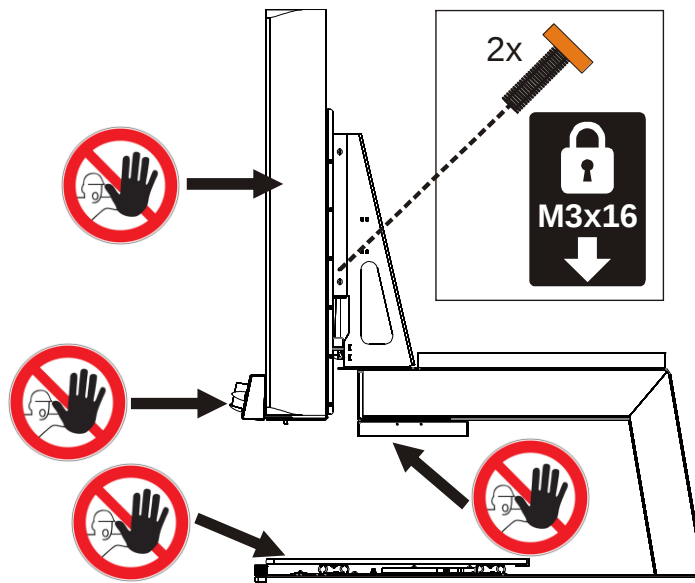

Maintenance instructions


- Occasionally clean the screen and the lens using a damp cloth. For further instructions: check the screen manual.
- Treat your device with care and you will enjoy it for years to come.

Chapter 2 : Installation

The Vocatex is solely to be installed by authorised personnel. You can find a list on www.kobavision.be

Chapter 3 : Controls

Operating panel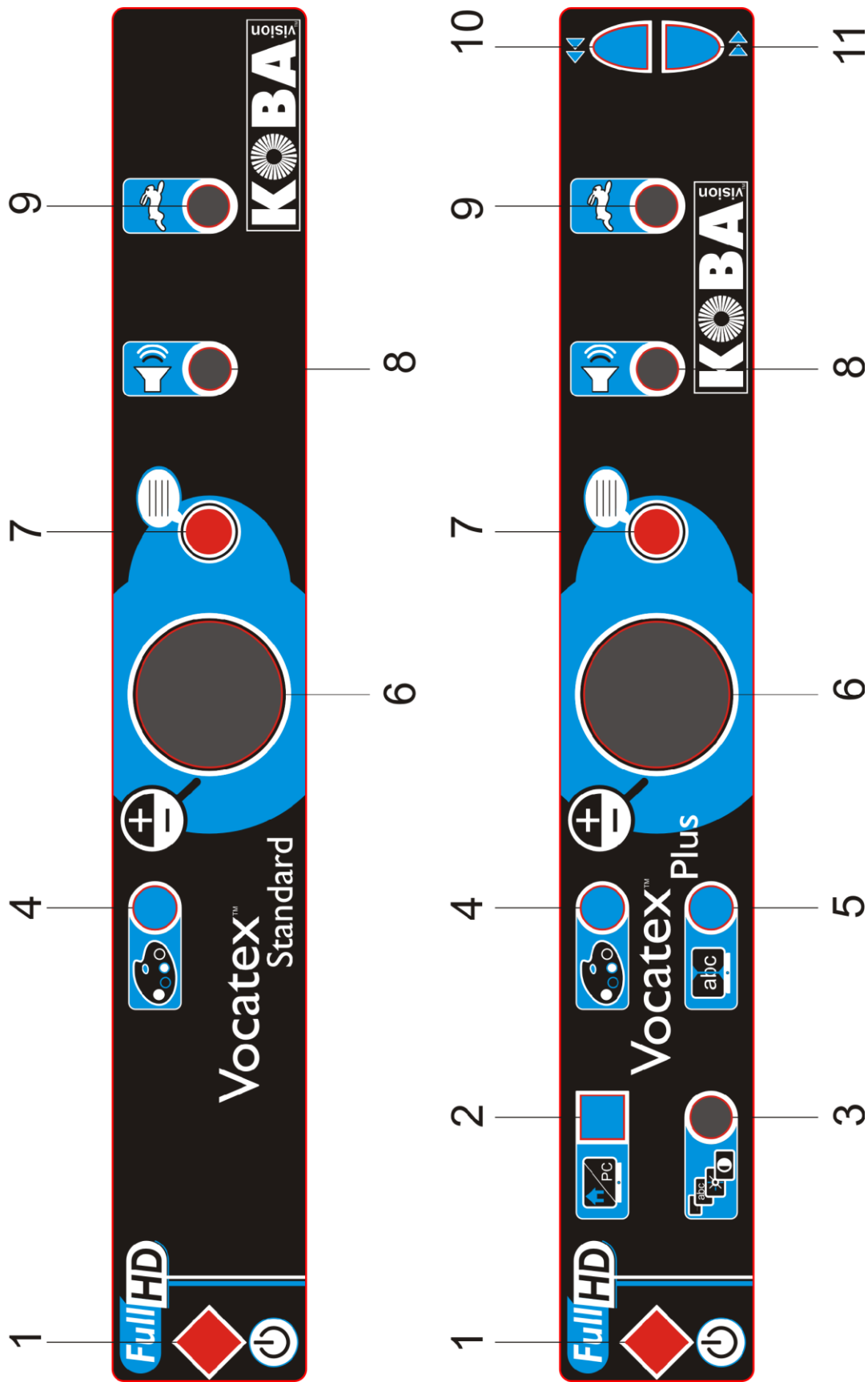

1	ON/OFF switch
2	Switch computer/Vocatex
3	Press: Switch between brightness and ticker Turn in colour mode: brightness Turn in positive or negative reading : improve background Turn when ticker is active: changes the height of the ticker Press and turn when the ticker is active: changes the position of the ticker
4	Press briefly: switch between photo mode, negative, positive reading Hold button: changes colour of the background or characters
5	Press briefly: hide/unhide column selector Hold button: change the colour of the column selector
6	Turn: zoom in/zoom out Press: overview/magnification
7	Read aloud
8	Turn: volume Press and turn: select another reading voice
9	Reading speed
10	Press briefly: go back one word in the text Hold button: go back one sentence in the text
11	Press briefly: go ahead one word in the text Hold button: go ahead one sentence in the text


Switching on/off

The device is switched on with **button 1**. Most screens are switched on by the Vocatex. If this is not the case, switch on the screen as well (see screen manual).

After switching on, an on-screen bar shows the start-up progress. The Vocatex indicates when it is ready to be used by telling “the device is ready for use”.

Procedure

1. Put the text on the sliding table beneath the Vocatex.
2. The text now becomes visible on the screen.
3. Select a text block by placing the text block in the middle of the screen or by crossing the text block by the column selector.
4. Press **button 7** to start the read aloud function.

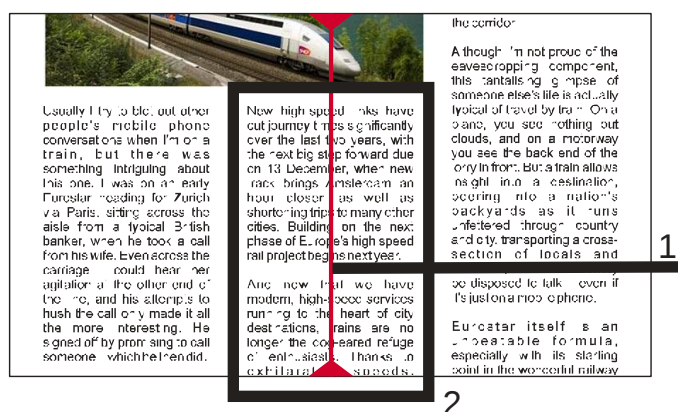1. Column Selector
2. Text block

Pause

Reading aloud is initiated by pressing **button 7**. To pause the text, you press **button 7**. To resume reading, you press **button 7**. If you do not move the text, the device will start reading from the last position.

Continuous reading

The Vocatex reads continuously. When you move further into the text, the Vocatex will continue reading without interruption. The reading frame disappears when moving but will reappear when the new position is found.

Navigation

The Vocatex Plus can navigate by word or by sentence. When pressing and holding **button 10** or **button 11** the device will go further or go back sentence by sentence. By pressing briefly you go further or go back word by word.

You can easily have the text read out again by sliding the text upwards again with the slide table. If you slide the text upwards by more than 1cm, the Vocatex will resume reading aloud from the start and not from the previous reading position.

Progress indicator

While reading, the Vocatex displays a progress indicator. This is a bar which is shown at the bottom of the screen and indicates the progress in the text. The progress indicator can appear in 3 different forms.

Reading within the same column

- Sliding up or continue reading without sliding the text that is being read out of screen: the reading frame disappears when the movement is stopped and reappears when the new position is found.

- Continue reading while the text is slid out at the top of the screen. The frame disappears and reappears when the text that is being read enters the screen again. When the text connects, this is shown by double bars that approach one and other.

- When it is impossible to reconnect, meaning that the text was slid through too far, an arrow appears to the right of the bar. You can position the text back until you see the indicator with the 2 bars.

Reading a next column

During the reading of one column you can already move to the next without interrupting reading of the previous column. In that case an arrow is shown next to the bar. To repeat a text you can slide the text downwards. The text will be read from the top.

Language and reading voices

You can change the language options at any time. To open the language menu, you press **button 7** for 10 seconds. The language menu will open. The menu has 3 steps:

Step 1: Language of the spoken assisting messages

This is where you adjust the language of the spoken buttons. You choose your mother tongue. An inhabitant of France chooses French. By turning the big knob you choose the language. By holding **button 7** you will go to step 2.

Step 2: Preferred voice


Most languages have several voices. In step 2 you choose your preferred voice for the spoken buttons and your preferred voice for reading out the texts. Usually you can choose between a male and female voice. In French, for example, you can choose between Virginie, Thomas, Felix and Audry. You select the language by turning the big knob. By holding **button 7** you will go to step 3.

Step 3: Other reading voices

If you would like to have texts read out by other voices or in different languages, you can select other voices in step 3. By turning the knob you select a voice. By briefly pressing **button 7** you can switch a voice on or off. (X) indicates that a voice is switched on. () indicates that a voice is switched off. You can switch on/off various voices. By holding **button 7** you will exit the voice and language menu.


Slide table

Under the table you can find a lever which controls the brake. If you move the lever to the left, the slide table is free to move left/right/up/down. If you move the lever to the right you adjust the friction. If you move the lever all the way to the right, the slide table will be blocked.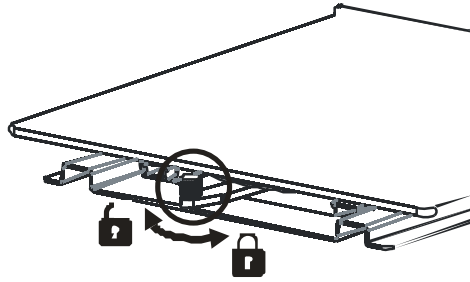

Height adjustment

According to personal preference, the screen of the Vocatex can be moved up or down. To do this, make sure you take hold of the screen on both the left and right side of the screen.

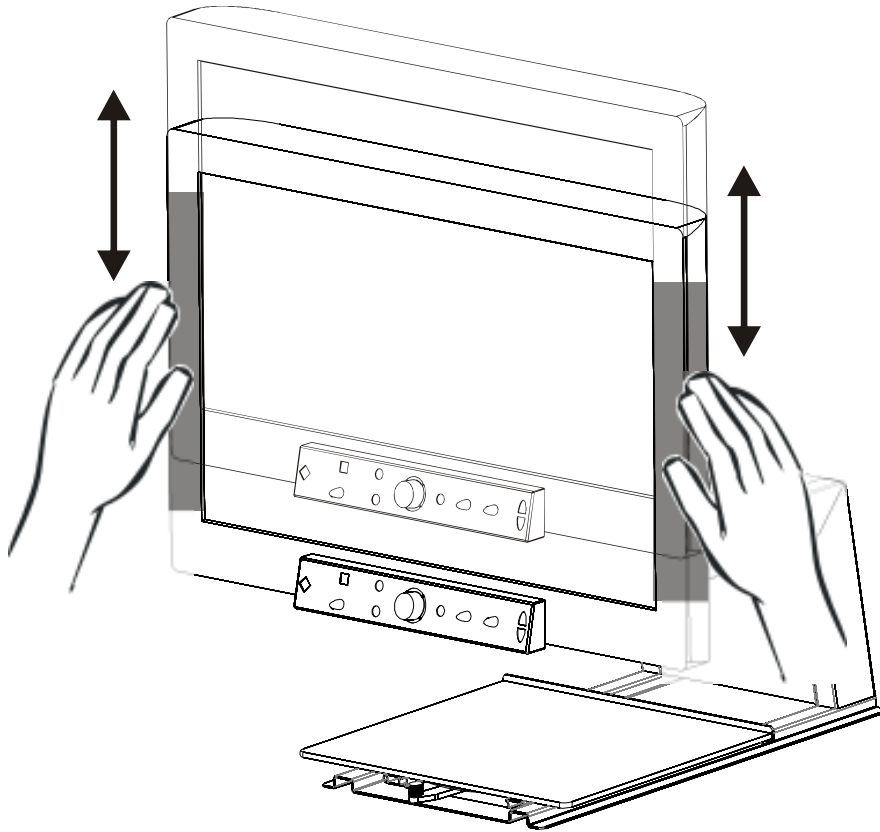Programmable buttons

The Vocatex has programmable buttons. **Buttons 2, 3, 4, 6** and the optional **foot switch** can have different functions. These functions can be activated by your dealer.

Button 2: Reset, Switch between magnifier and computer

Button 3: Brightness/background, Lighting, Reading line, Reading bar

Button 4: Different colour combinations

Button 6: Overview/magnification, Switch between magnifier and computer, Reset, Reading aloud

Foot switch: Overview/ magnification, Switch between magnifier and computer, Reset, Reading aloud

Pitch control

Press the volume button (**button 8**) and turn the speed button at the same time (**button 9**).

Split screen

The Vocatex Plus has a split screen connection. Contact your dealer for more information.

Usage tips

Reading aloud – Text blocks

Reading aloud is best done in a reading mode where you easily can select the text blocks.

Reading aloud – Recognition

Text recognition isn't 100% accurate. The devices will make mistakes. By changing the magnification you can adapt the quality of recognition.

Reading

Reading is best done in positive or negative mode. These modes adapt the image for the visually impaired to a high-contrast image with improved background. Background and letters can be set at your discretion. The background can be set by means of background suppression. This will allow you to filter out the structure of a sheet. This unique KOBA Vision function allows you at all times to set a contrasty background without annoying paper impurities. A newspaper can be read without annoying paper impurities. Factory settings will take you back to the original settings.

Watch photos

Photos are best watched in photo mode. This mode is a unprocessed image without contrast enhancement. Using the buttons contrast, brightness and colour, you can adjust the image so that the photo becomes clearly visible. Factory settings will take you back to the original settings.

Crafting

Crafting is best done with a small enlargement. It is better to select a dark background during this activity so that the camera is not hindered by large differences in contrast with whatever is darkened by the hands.

Writing

You can also write under your video magnifier. It is best to use a black pen and lined paper.

Chapter 4 : Troubleshooting

Below is a list of problems that may occur with your Vocatex. Consult the list first before contacting KOBA Vision or your dealer.

Symptom	Possible causes	What to do
No display on the screen	• Power failure • The screen is not switched on	• Check whether the plug of the power cord has been properly plugged into the wall socket. If the device still won't work, pull the plug out of the wall socket. Then wait 60 seconds before plugging it back into the wall socket. Turn the screen on again. • Press the "on" button on the remote
No sound	• The screen volume is on zero • The sound has been disrupted • The headphone is plugged in • The HDMI connector is loose	• Slightly turn up the screen volume. • Check whether the sound has not been disrupted. • Point the remote control directly towards the sensor at the front of the screen. • Unplug the headphone • Check the HDMI plug.

No sound or image while the red stand-by light at the front of the screen is burning	• Screen is in stand-by mode	• Press the “on” or “1” button on the remote
The pc modes don't work	• Wrong connectors • Wrong source • Wrong screen resolution	• Check whether the source has been selected correctly. Press the button to open the source list and select the correct source. • Check whether you have set the computer to the correct screen resolution. • Check the settings of the video card in the pc.

Chapter 5 : Warranty

Express warranty

KOBA Vision bvba offers a two-year warranty on every KOBA VISION video magnifier, starting from the purchase date. The warranty covers materials and workmanship. To be able to benefit from the warranty, you need to keep the box and the packaging material.

Warranty services are provided by KOBA Vision bvba or dealers acting on behalf of KOBA Vision bvba. When the video magnifier cannot be repaired correctly, KOBA Vision reserves the right to replace the reading device by an equal device. All components and products replaced within the warranty period become property of KOBA Vision bvba.

Warranty provisions

1. The warranty becomes invalid if the video magnifier:

- Was not used according to instructions or was misused,
- Was damaged as a result of a fall or impact,
- Was disassembled or tinkered with,
- Was repaired earlier by a non-official service partner.

2. Consequential loss is under no circumstances covered by the warranty.

KOBA Vision bvba will neither assume any liability whatsoever nor accept any possibility of liability except for the warranty provisions explicitly stated above. This limitation to the liability does not imply a breach of the customer's legal consumer rights or his or her rights vis-à-vis the vendor.

Service fulfilment

Before claiming the warranty or service and before contacting KOBA Vision bvba or a dealer, you must note product name, serial number, purchase date, invoice number and any error messages or particular problem aspects.

Chapter 6 : Specifications

Type:	KOBA Vision Vocatex Standard KOBA Vision Vocatex Plus
Inputs:	HDMI OUT, POWER IN, 2x USB
Power:	Input: 12V DC 3,75A (MAX) Max. Power: 45W Standby Power: 0,16W
Dimensions (without screen): H X W X D	270 x 415 x 560 mm
Maximum viewing angle:	220 mm
Minimum viewing angle:	9 mm
Magnification:	<i>Vocatex Standard 22" – 58cm</i> 3x – 56x <i>Vocatex Plus 26" – 66cm</i> 2,6x – 64x <i>Vocatex Plus 32" – 82cm</i> 3,2x – 78x
Vocatex Standard: • Red rectangle: word and text block • Column selector • Continuous reading • Updating word frame • Progress indicator • Live mode • Reading speed • Volume • 1 language • Photo mode • 16 colour combinations • Autofocus • Led lighting • Spoken buttons • FULL HIGH DEFINITION Vocatex Plus: • Red rectangle: word and text block • Column selector • Continuous reading • Updating word frame • Progress indicator • Live mode • Reading speed • Volume • Multiple languages • Photo mode • 28 colour combinations • Autofocus • Led lighting • Spoken buttons • FULL HIGH DEFINITION • Contrast: 5 levels • PC-connection • Navigation: next/previous sentence and word • Programmable buttons: brightness/background, lighting, reading line, reading bar	

© **Kوبا Vision** • De Oude Hoeven 6 • 3971 LEOPOLDSBURG • Belgium
 Tel +32 11 51 70 80 • Fax +32 11 51 70 81 • info@kobavision.be
<http://www.kobavision.be>

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